

How to Choose the Correct Size?

VACOCast Diabetic is a standardised product which can be adapted very easily to different foot shapes by means of OPED's vacuum technology. At the same time, this standardisation results in certain minimum and maximum patient foot sizes for which the VACOCast Diabetic can be applied.

VACOCast Diabetic is available in size small, medium and large which cover 95% of all patients.



You can compare the patient's foot measurements with the following table to find the suitable size:

VACOCast Diabetic	Small	Medium	Large
Calf circumference	23 - 39 cm	35 - 51 cm	40 - 58 cm
Ankle circumference	14 - 29 cm	22 - 40 cm	30 - 45 cm
Foot width at ball	10.5 cm	12 cm	13.5 cm
Shoe sizes (for orientation purposes only)	2 ½ - 5 ½	5 ½ - 9	9 - 14

There may be deviations in this classification in the case of extreme deformities.

VACOCast Diabetic Order Codes

VACOCast Diabetic	Small	Medium	Large
Item code	VPCD-01-1-S	VPCD-01-1-M	VPCD-01-1-L
PIP code	413-0522	413-0530	413-0514
GTIN	04061457000179	04061457000162	04061457000155

OPED UK Ltd
Unit 5
Avro Business Centre
Avro Way
Bowerhill
Melksham
SN12 6TP
United Kingdom

Phone +44 (0)1380 722177
Fax +44 (0)1380 710095
enquiries@oped.biz
www.oped-uk.com



Keeps you going.